


ReLax Installation Procedure

I. Download APP from the website


1. Create file on desktop (recommend naming the folder "OPI").
2. Enter www.op-innovations.com website. Find "APP Corner" on menu bar and download "ReLax...App". Place downloaded APP into the new folder and save.



3. Decompress the downloaded file.
 - a. Open OPI folder;

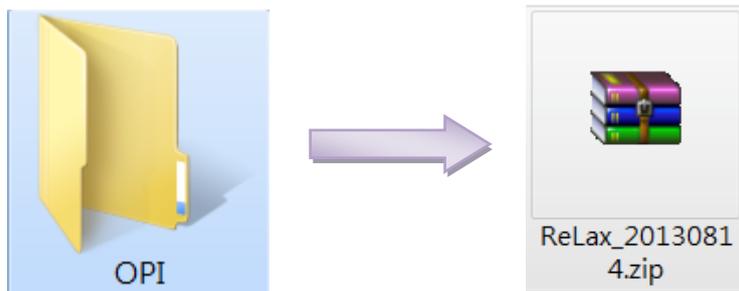

- b. Extract files;

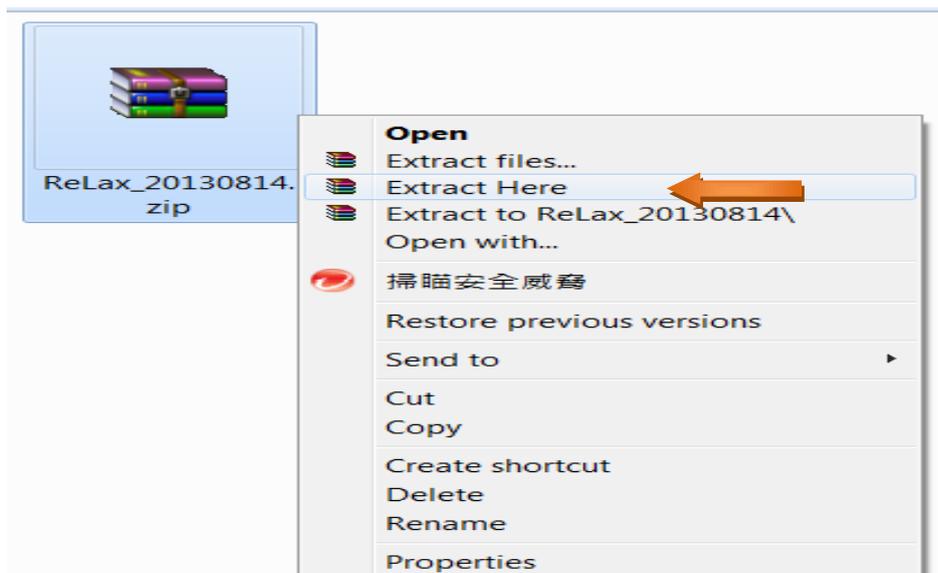


- c. All files are located in OPI folder.


II. Installation of the device driver

1. Plug the device (Bio-sensor and Controller) into the computer USB slot. (Note: Yellow battery should be facing opposite each other)


Click the "Start" menu button → "Control Panel" → "Hardware and Sound" (or "Device Manager") → "Device Manager".

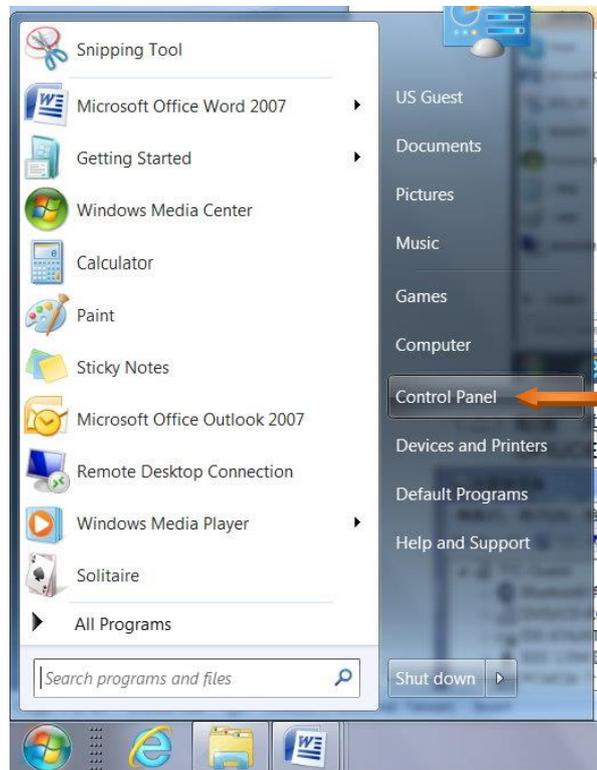


2. Double click the "Other devices"; the device name appears as "OPIUCE", double click "OPIUCE".

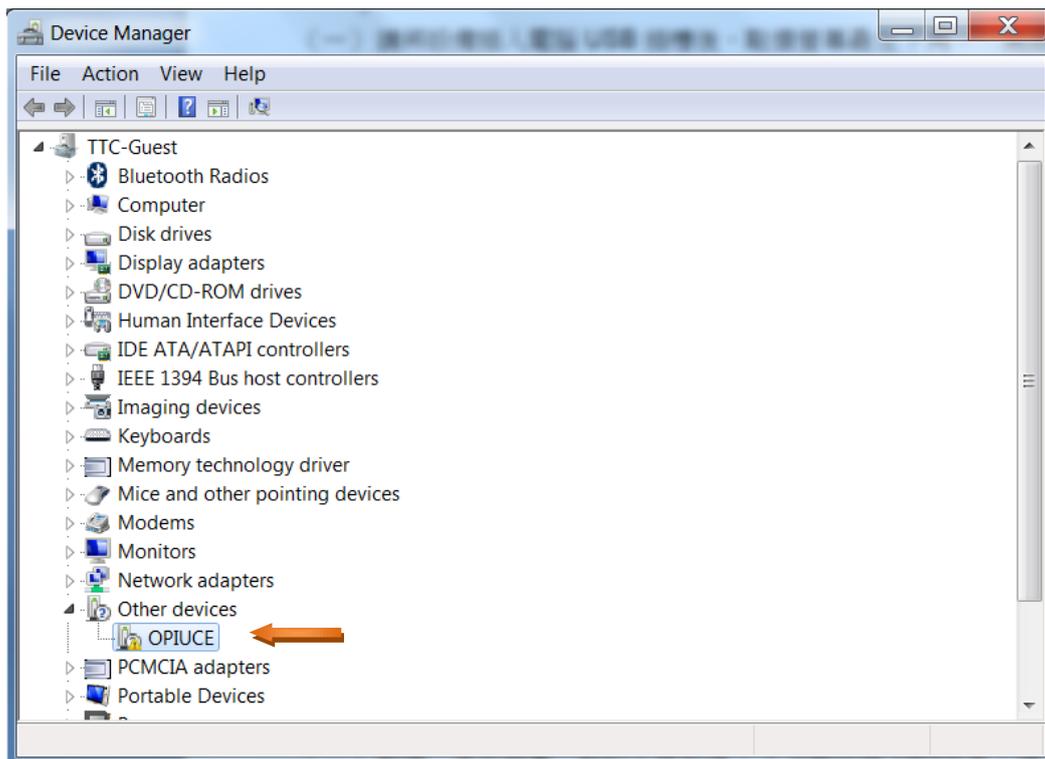


3. Choose "Driver" → click "Update-Driver" → click "OK".

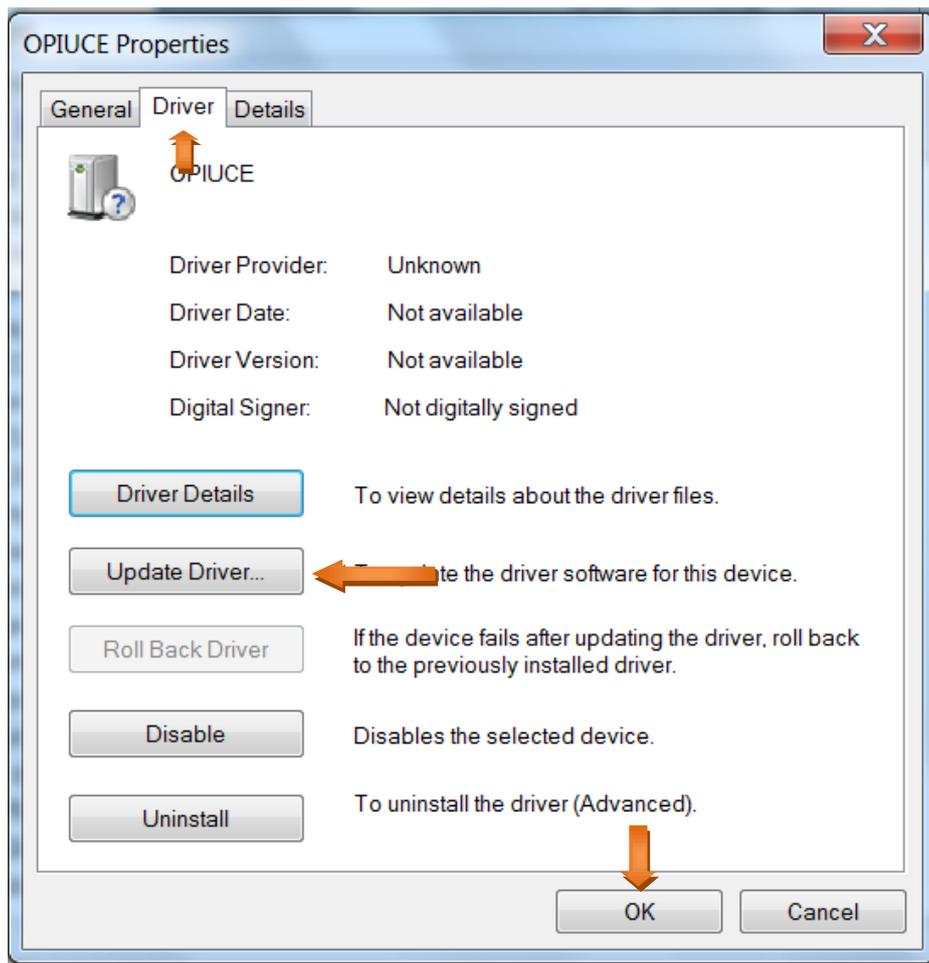

4. Click "Browse my computer for driver software".

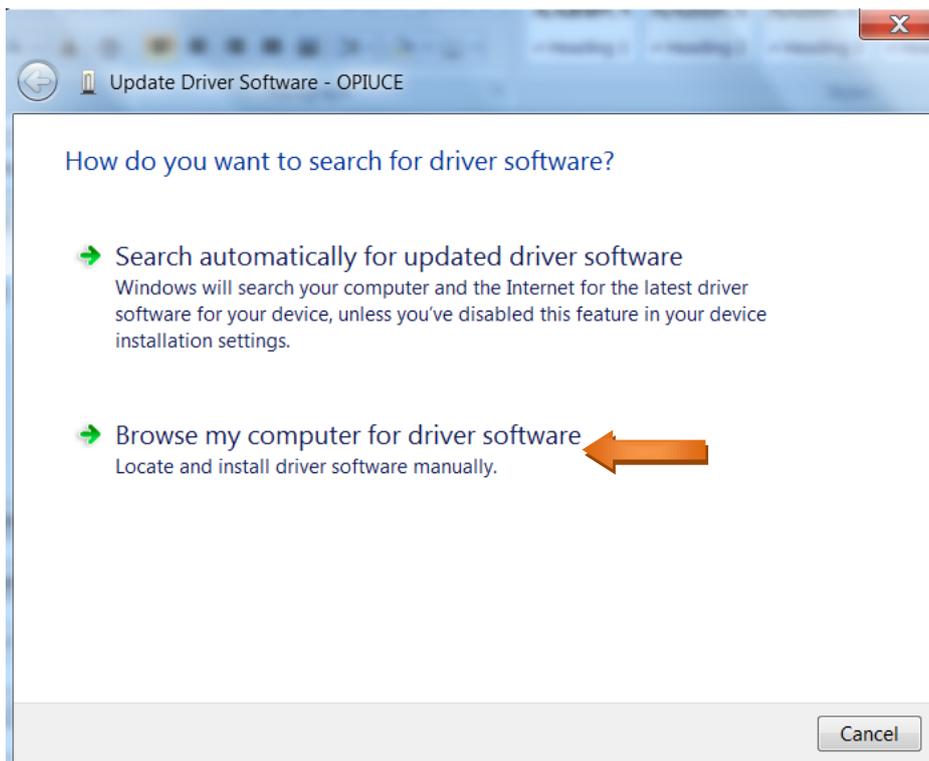


5. Click "Browse..." → open OPI folder, select "win_usbedc_driver" folder → click "Next".

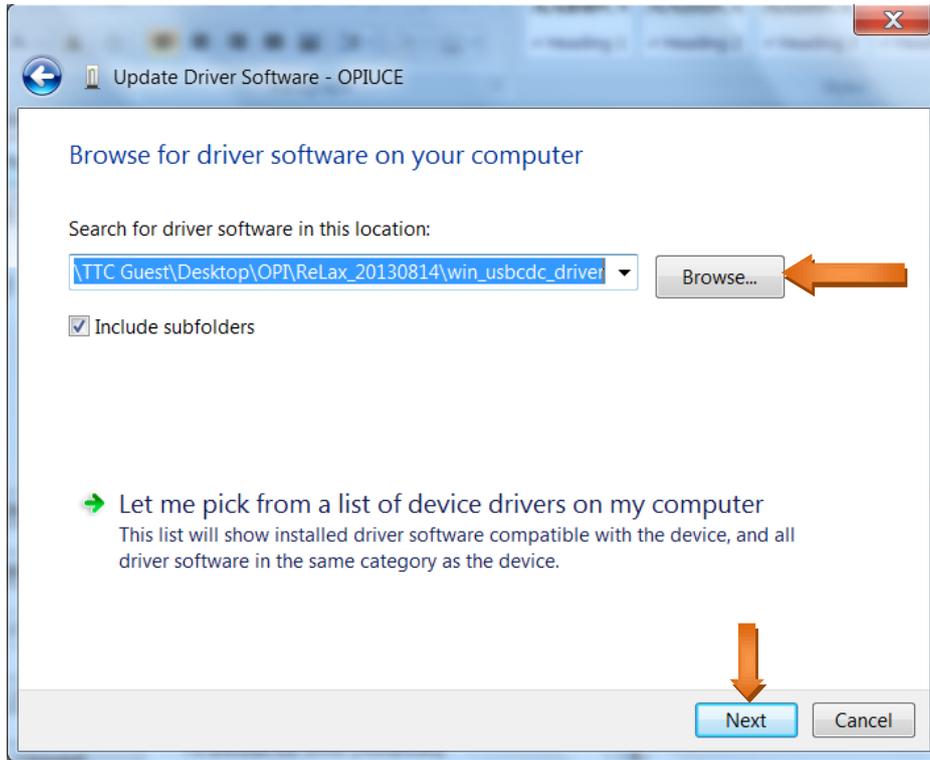


Installation is in process...wait.

6. When the following screen appears, the driver is successfully installed.

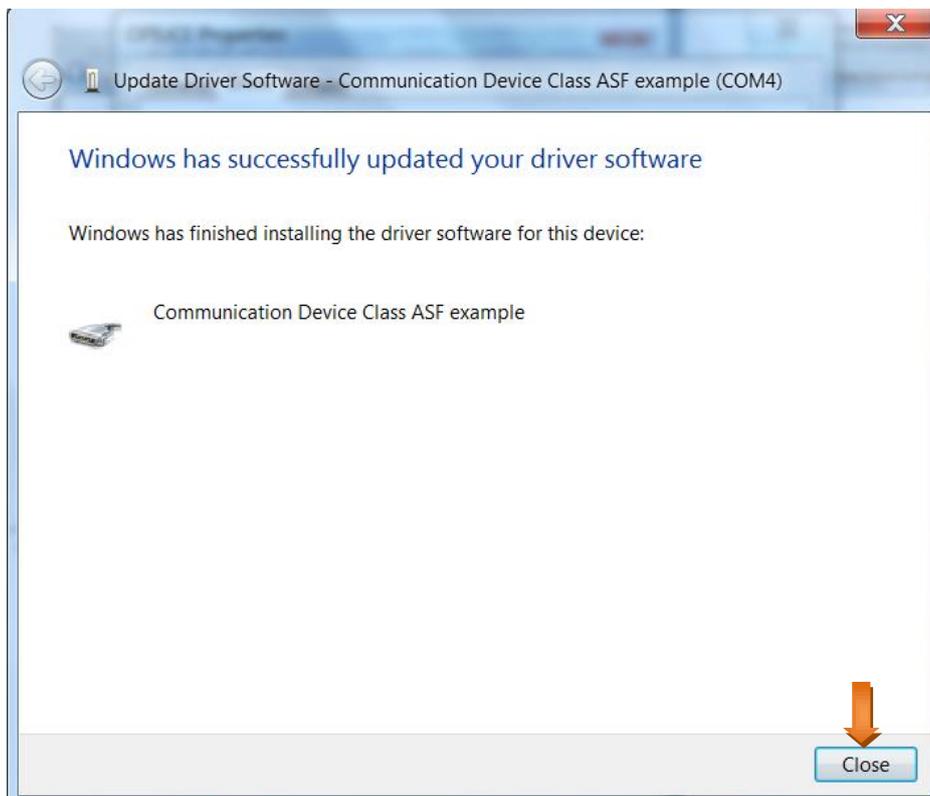


III. Run the application program


1. Open OPI folder, then open the “ReLax.....exe” folder. (Video Instruction is also provided in www.op-innovations.com website.)

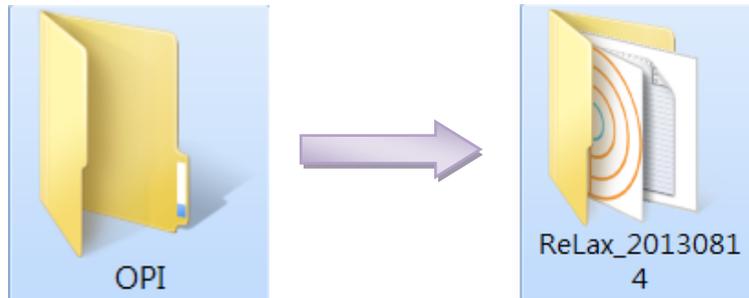


2. With the device (Bio-sensor and Controller) plugged in, run “ReLax.....exe” program.

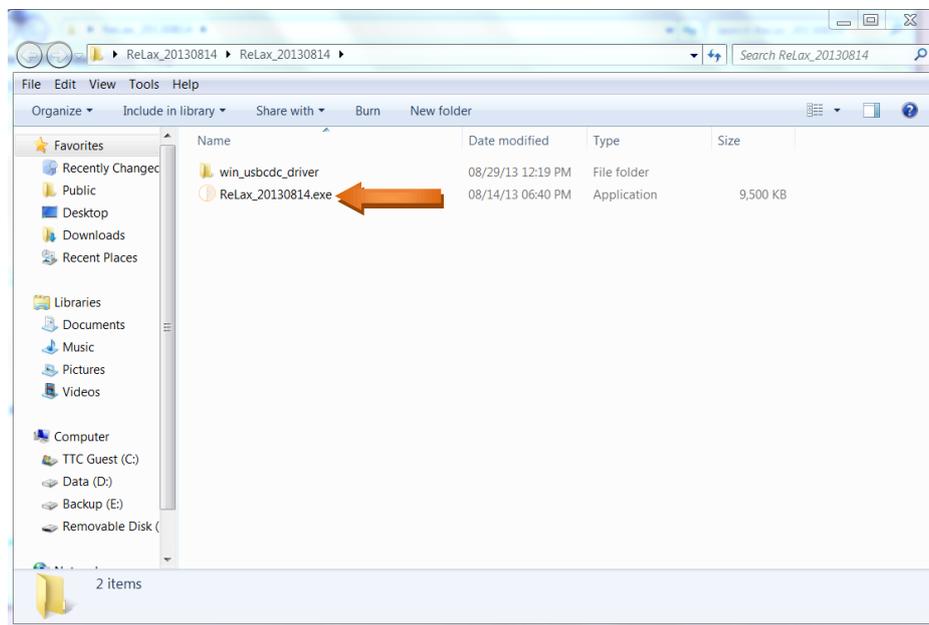

3. Execution screen.

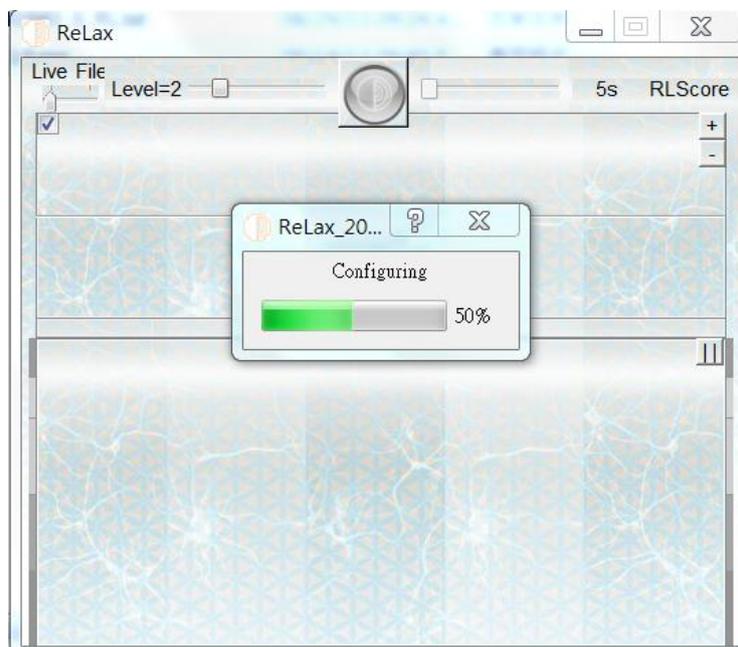

4. Choose level. Remove Bio-sensor from the Controller (which remains plugged into the computer). Place headband on a flat surface and attach sensor to the headband “snaps”. Make sure the “silver buttons” are on the middle of forehead. Press “Start” button.

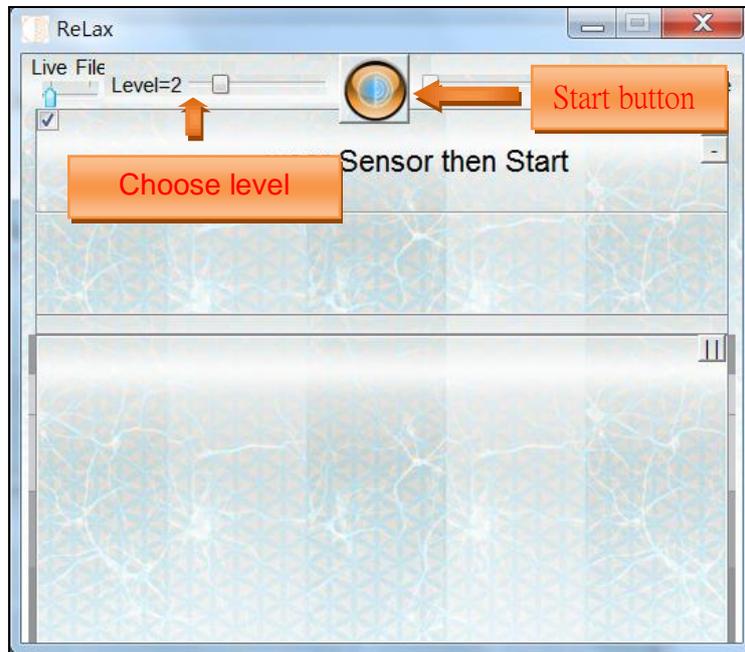

IV. Screen showing the status of the activity

The status of monitoring is shown on the screen.

