

TrueSense Installation Procedure


I. Download APP from the website

1. Create file on desktop (recommend naming the folder "OPI").
2. Enter www.op-innovations.com website. Find "APP Corner" on menu bar and download "OPI Console and SDK download...App". Place downloaded APP into the new folder and save.




3. Decompress the downloaded file.

- a. Open OPI folder;

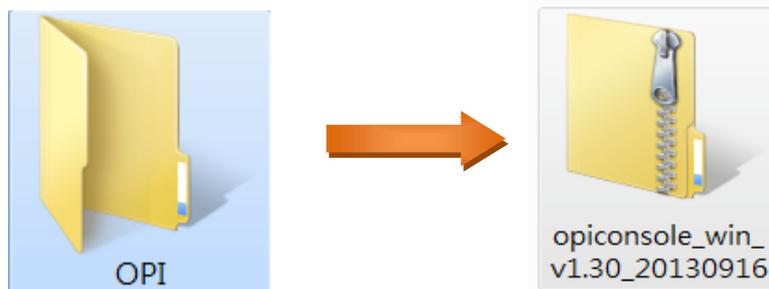

- b. Extract files;

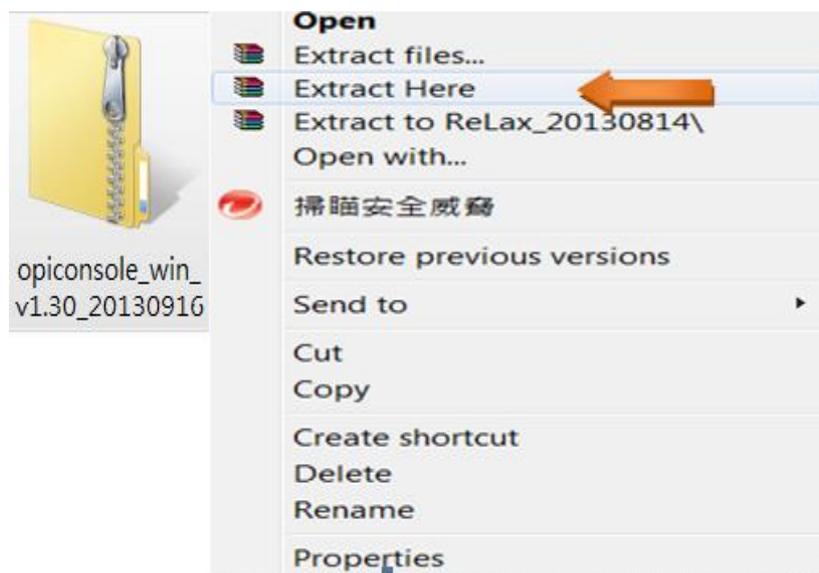


- c. All files are located in OPI folder.

II. Installation of the device driver

1. Plug the device (with Bio-sensor, memory and Controller connected) into the computer USB slot. (Note: batteries facing same direction)

Click the “Start” menu button → “Control Panel” → “Hardware and Sound” (or “Device Manager”) → “Device Manager”.

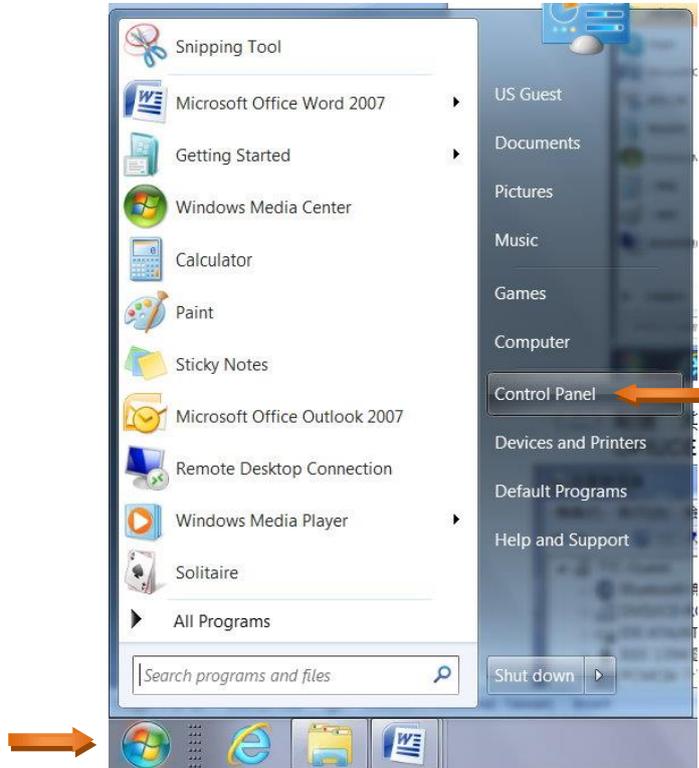

Note: Illustration of Connection


- (1) When Sensor, Memory and Controller are connected: Batteries are in the same direction.




- (2) When Sensor and Controller are connected: Batteries are in the opposite direction.




2. Double click the “Other devices”; the device name appears as “OPIUCE”, double click “OPIUCE”.

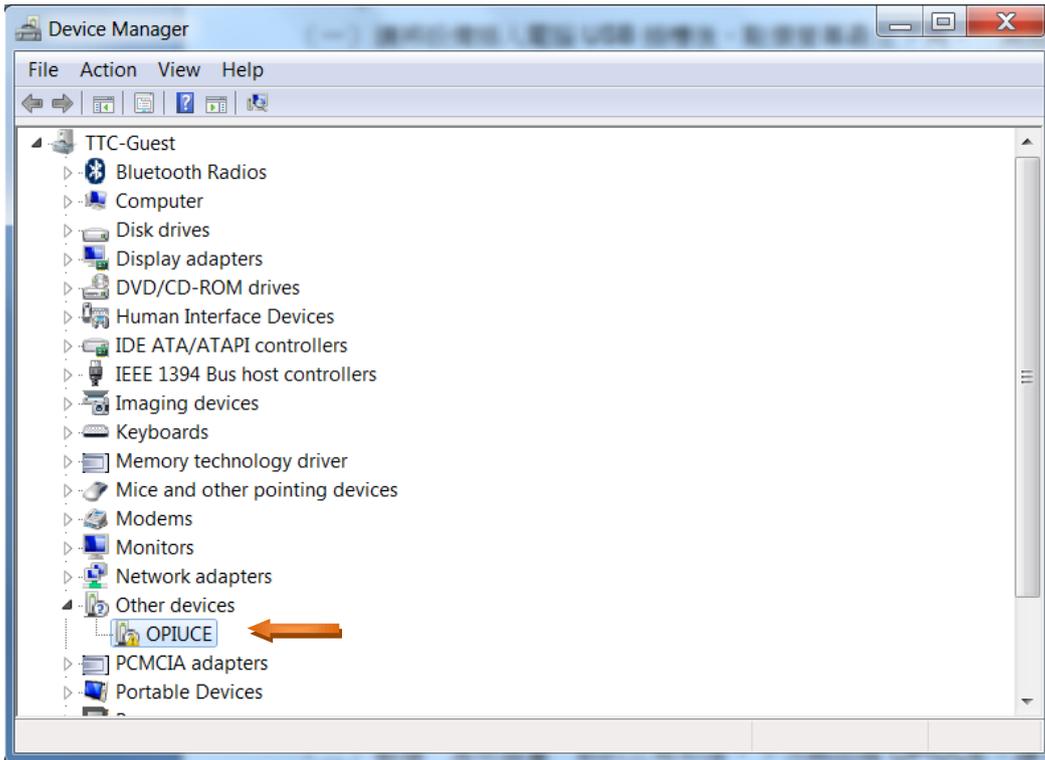


3. Choose “Driver” → click “Update-Driver” → click “OK”.

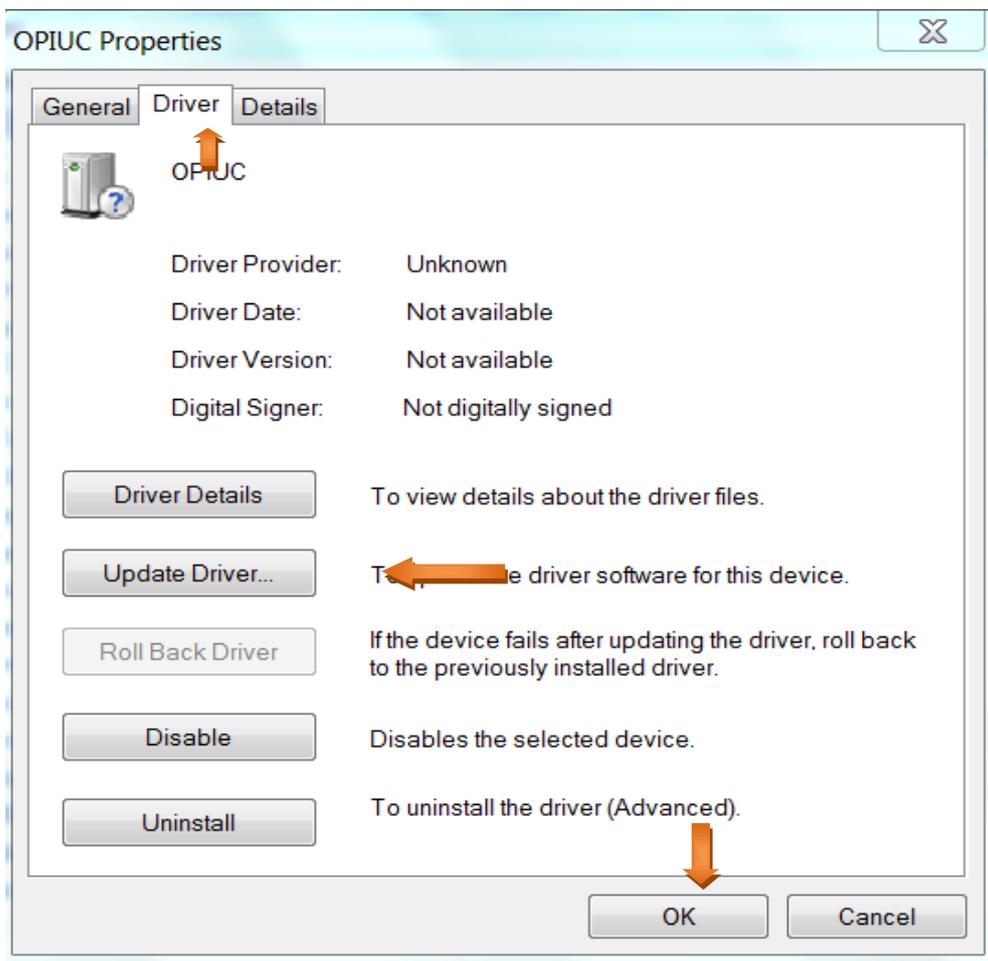

4. Click "Browse my computer for driver software".

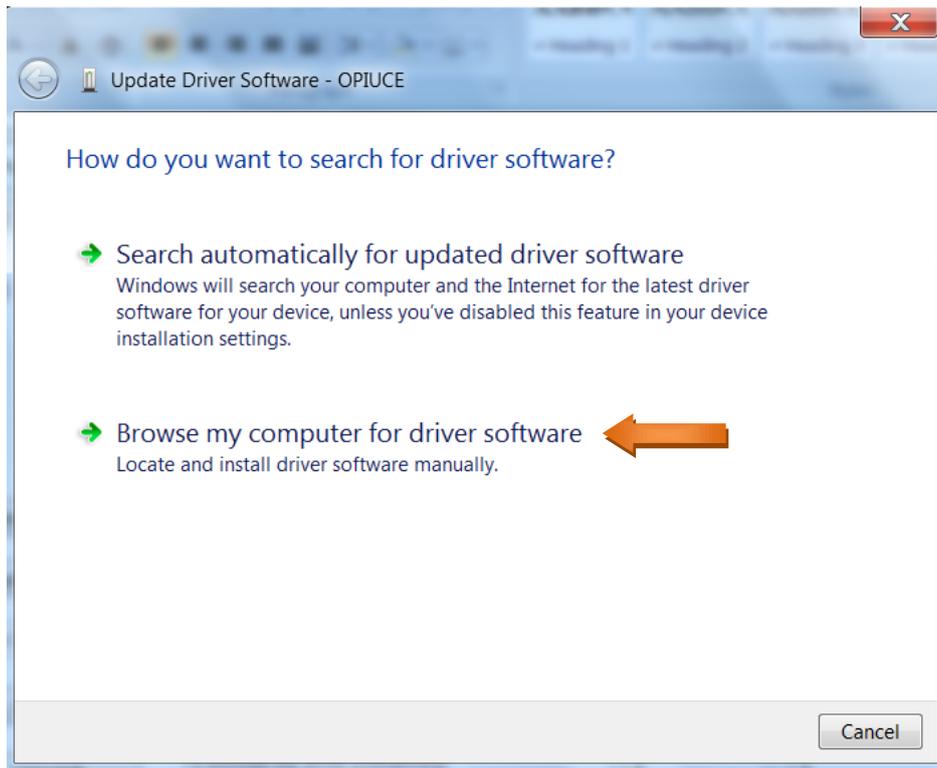


5. Click "Browse..." → open OPI folder, select "win_usbedc_driver" folder → click "Next".

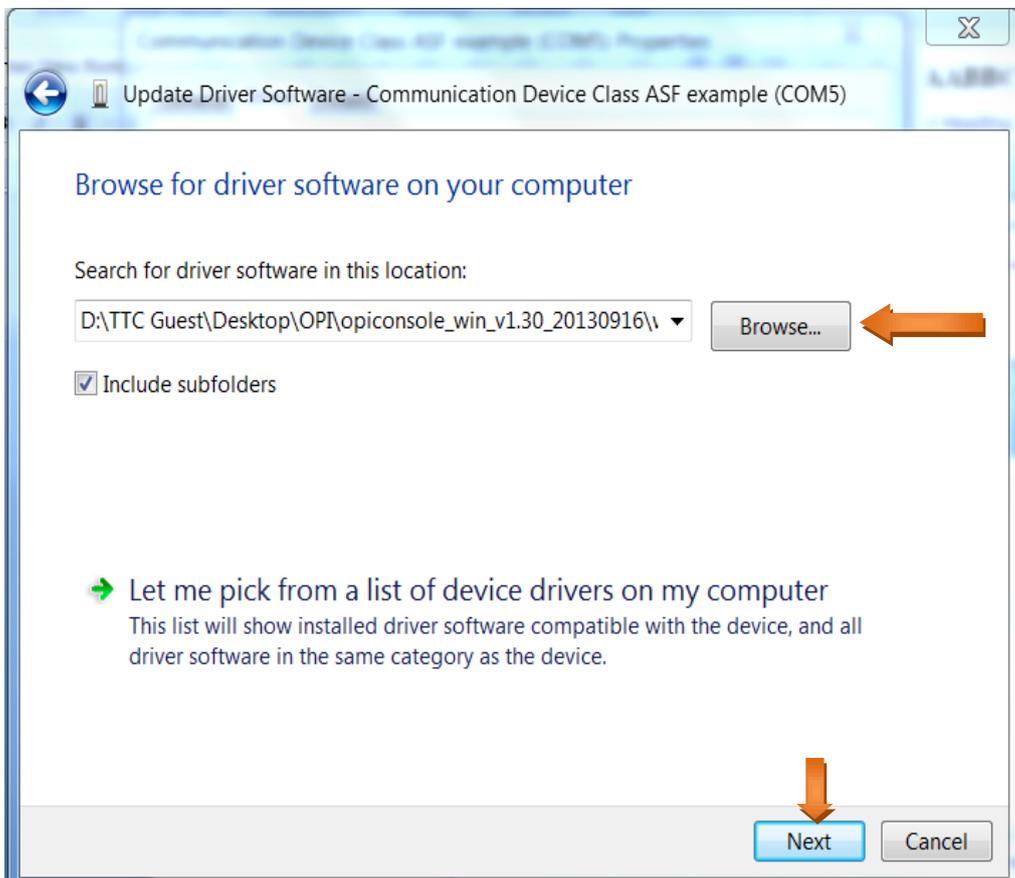


Installation is in process...wait.

6. When the following screen appears, the driver is successfully installed.

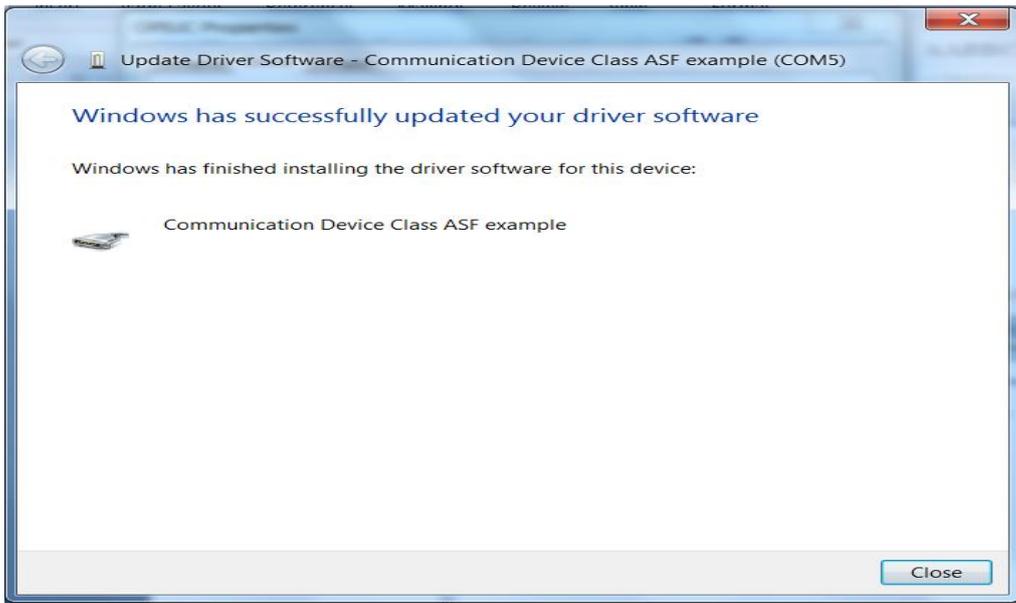


III. Run the application program

1. Open OPI folder, then open the “opiconsole_win_.....exe” folder. (Video Instruction is also provided in www.op-innovations.com website.)

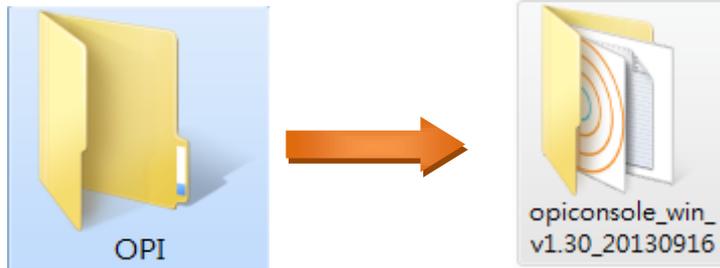

2. With the device still plugged in (with Bio-sensor, memory and Controller connected), run “OPIconsole.....exe” program.

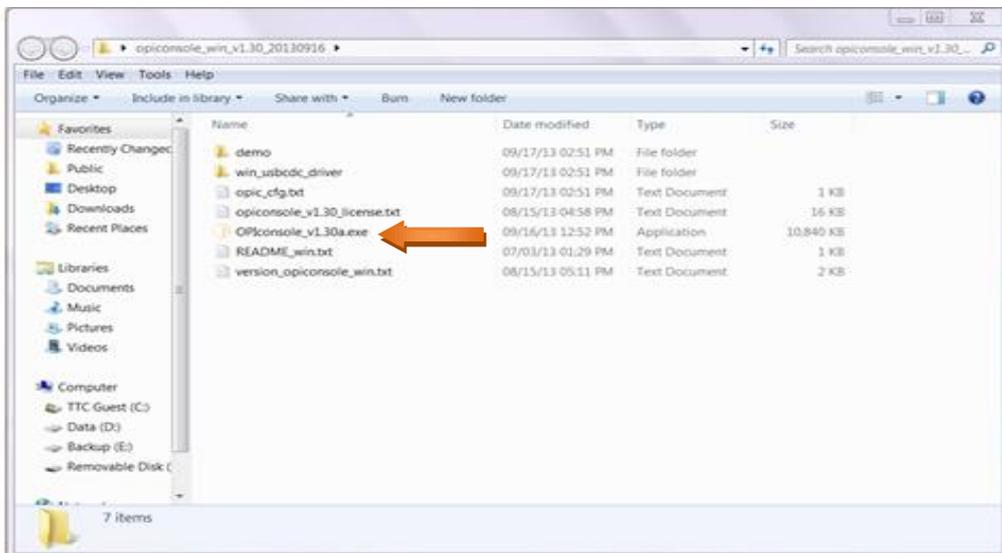

3. Execution screen.

